


DOCTOR INSTRUCTIONS

Thank you for purchasing the Vision Assessment Corporation Zoo Polarized Variable Vectograph (P/N 1037).

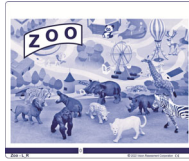

PURPOSE

A three-dimensional picture is used to strengthen the binocularity system and provide base-in and/or base-out training.

FAMILIARIZE YOURSELF WITH THE VECTOGRAPH

- Polarized Variable Vectograph Vision Therapy System consists of:

1. 1 Therapy Binder



2. 1 Guide



3. 2 Vectographic Panels

- Panel 1 (Blue Bar at Bottom)



- Panel 2 (Numbers/Letters at Bottom)



4. 1 Pair Standard Polarized Viewers



(NOT TO BE USED AS SUNGLASSES)

5. 2 Instruction Manuals

- Doctor Instruction Manual
- Patient Instruction Manual

This exercise works best if practiced daily as suggested. A Patient Instruction Manual has been included so the Vectograph Vision Therapy System can be sent home with the patient to allow for greater success through continued at-home usage and practice.

6. 1 Patient Vision Therapy Record Form with Pen

TESTING CONDITIONS

- Well-lit, glare-free area
- If reflections or glare on the Vectograph can be seen, try tilting the test or choose another testing location

ADMINISTRATION

1. Place the polarized viewers on the patient.

PLEASE NOTE: DOCTOR SHOULD ADVISE WHETHER OR NOT POLARIZED VIEWERS SHOULD BE WORN OVER PATIENT'S PRESCRIPTION GLASSES.

2. Hold the Vectograph approximately 16 inches (40 cm) in front of the patient.
3. Slide the 2 panels in the guide until the 0 appears in the opening on the blue bar at the bottom of the test.
4. Each of the Zoo animals are arranged at different levels of depth with varying seconds of arc. Ask the patient to point to/identify each animal and indicate which animals seem to be floating away from them (BELOW the Vectograph), towards them (ABOVE the Vectograph) or if the animal appears EVEN with the Vectograph. Ask the patient to indicate which animals appear closer and which appear further away.
5. Refer to the "Answer Key" section of this manual for the seconds of arc and direction of float for each animal.
6. Next, while slowly slide the panels apart ask the patient to try to maintain a single image while repeating Step 5.
7. Have the patient notify you when the images double or when he/she is no longer able to fuse the Zoo images. (This is the Breaking Point.)
8. Note the number/letter in the space on the blue bar at the bottom of the Vectograph. Refer to "Scoring" section of this manual.
9. Next, as you slowly slide the panels back together, ask the patient to notify you when he/she can see the Zoo images as single clear images again. (This is the Recovery Point.)
10. Note the number/letter in the in the space on the blue bar at the bottom of the Vectograph. Refer to "Scoring" section of this manual.
11. Repeat steps 7 – 11 until patient can achieve the doctor's recommended goal.
12. Follow any variation on these exercises as recommended by the doctor.

SCORING

- Each letter A-P on the bottom blue bar represents one diopter (Base-In) (Divergence / Relaxing).
- Each number 1-10 on the bottom blue bar represents one diopter. (Base-Out) (Convergence / Crossing).
- Each number 10-24 represents two diopters. (Base-Out) (Convergence / Crossing).
- 40 diopter range of separation available.
- Refer to the Patient Vision Therapy Record Form.
- Record the date.
- Above the diagonal line in the row labeled "Relaxing" record the Breaking Point for the highest letter on the blue bar that the patient is able to achieve.
- In the same box, below the diagonal line, record the Recovery Point for the highest letter from the blue bar at which the patient is able to fuse the images as a single clear image again.
- Repeat the previous 2 steps for the row labeled "Crossing" for recording the Breaking Point and Recovery Point for the highest numbers that the patient can reach on the blue bar at the bottom of the Vectograph.

ANSWER KEY

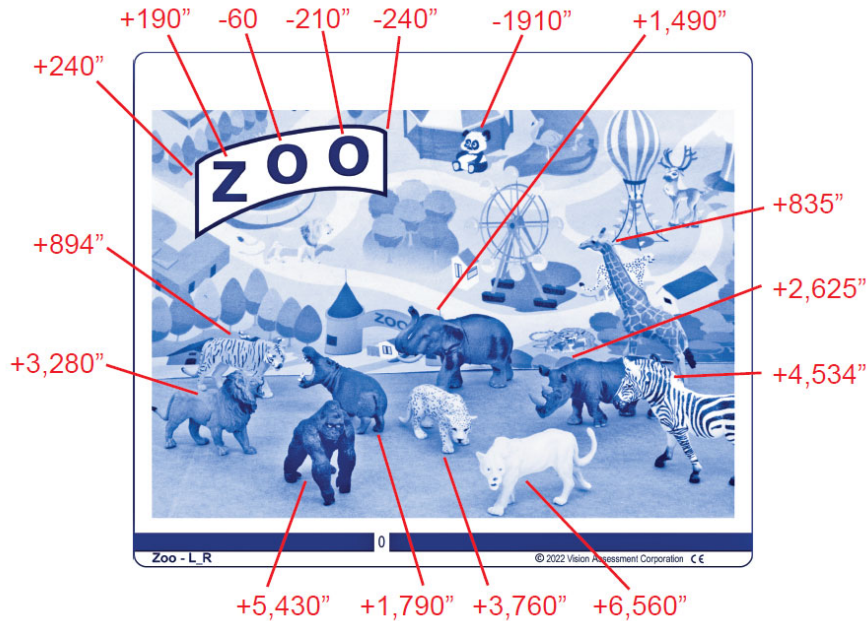


Image Orientation and Stereopsis Levels in Seconds of Arc (Clockwise Left to Right):

The higher the seconds of arc, the greater the float ABOVE the Vectograph

The more negative the seconds of arc, the further the float BELOW the Vectograph

Cougar: ABOVE: 6,560"	Panda: BELOW: -1910"
Cheetah: ABOVE: 3,760"	Elephant: ABOVE: 1490"
Hippopotamus: ABOVE: 1,790"	Giraffe: ABOVE: 835"
Gorilla: ABOVE: 5,430"	Rhinoceros: ABOVE: 2,625"
Lion: ABOVE: 3,280"	Zebra: ABOVE: 4,534"
Tiger: 894"	

CARE/HANDLING & STORAGE

- Clean vectographic panels and guide with a soft, damp, lint-free cloth. Dampen cloth using glass cleaner or mild detergent/water.
- **CAUTION: DO NOT IMMERSE THE VECTOGRAPHIC PANELS IN WATER. DO NOT SPRAY CLEANER DIRECTLY ONTO PANELS.**
- ☀️☔ Store Vectograph in a dry place away from direct sunlight.
- Clean polarized viewers using lens cleaner and soft, lint-free cloth.
- If panels are removed from guide during cleaning, replace the panels in the guide placing the panel with the blue bar on top of the panel with the numbers/letters. and ensuring that the plastic portion of the guide is behind the panels.

WARRANTY

- 1-year manufacturer warranty from date of purchase



Zoo

Polarized Variable Vectograph

P/N 1037

DOCTOR INSTRUCTIONS

Vision Assessment Corporation

2675 Coyle Avenue


Elk Grove Village, Illinois 60007 USA

Phone: 1 847 239 5889

Toll Free USA: 1 866 887 9692

Email: sales@VisionAssessment.com

Web: www.VisionAssessment.com

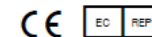

MDSS GmbH,
Schiffgraben 41, Hannover 30175, Germany