

PATIENT INSTRUCTIONS

Thank you for purchasing the Vision Assessment Corporation Pumpkin Patch Polarized Variable Vectograph (P/N 1047).



PURPOSE

A three-dimensional picture is used to strengthen the binocularity system and provide a global image for base-in and/or base-out training.

FAMILIARIZE YOURSELF WITH THE VECTOGRAPH

- Polarized Variable Vectograph Vision Therapy System consists of:

1. 1 Guide



2. 2 Vectographic Panels

- Panel 1 (Blue Bar at Bottom)



- Panel 2 (Numbers/Letters at Bottom)



3. 1 Pair Standard Polarized Viewers



(NOT TO BE USED AS SUNGLASSES)

4. 2 Instruction Manuals

- Doctor Instruction Manual
- Patient Instruction Manual

This exercise works best if practiced daily as suggested. A Patient Instruction Manual has been included so the Vectograph Vision Therapy System can be sent home with the patient to allow for greater success through continued at-home usage and practice.

5. 1 Patient Vision Therapy Record Form with Pen

TESTING CONDITIONS

- Well-lit, glare-free area
- If reflections or glare on the Vectograph can be seen, try tilting the test or choose another testing location

ADMINISTRATION

1. Place the polarized viewers on the patient.

PLEASE NOTE: DOCTOR SHOULD ADVISE WHETHER OR NOT POLARIZED VIEWERS SHOULD BE WORN OVER PATIENT'S PRESCRIPTION GLASSES.


2. Hold the Vectograph approximately 16 inches (40 cm) in front of the patient.
3. Slide the 2 panels in the guide until the 0 appears in the opening on the blue bar at the bottom of the test.
4. Begin by performing a suppression check. Ask the patient to look at the plus sign inside of the circle in the middle of the Vectograph image. The patient should be able to simultaneously see both the vertical line of the plus sign (seen with the right eye), and horizontal line of the plus sign (seen with the left eye) before proceeding with this exercise.
 - If the patient is not able to see/maintain the horizontal line of the Plus Sign, the patient is suppressing vision in his/her left eye. Instruct the patient to cover their right eye. Have the patient focus with their left eye until he/she is able see the horizontal line that was not seen when viewing the Vectograph binocularly. Then again have the patient view the Vectograph binocularly to ensure that the horizontal line and the complete plus sign can be seen.
 - If the patient is not able to see/maintain the vertical line of the Plus Sign, the patient is suppressing vision in his/her right eye. Instruct the patient to cover their left eye. Have the patient focus with their right eye until he/she is able see the vertical line of the Plus Sign that was not seen when viewing the Vectograph binocularly. Then again have the patient view the Vectograph binocularly to ensure that the Vertical line and complete plus sign can be seen.
5. Each of the images within the Vectograph are arranged at different levels of depth with varying seconds of arc. Ask the patient to point to/identify the images and indicate if each appears to be floating away from them (BELOW the Vectograph), towards them (ABOVE the Vectograph) or if it does not appear to float (EVEN) with the Vectograph).
6. Refer to the "Answer Key" section of this manual for the seconds of arc and direction and depth of float for each of the images. Ask the patient to compare various objects within the Vectograph and indicate which object is closer and which is further away.
7. Next, while slowly sliding the panels apart ask the patient to try to maintain a single image while repeating Steps 5 & 6.
8. Have the patient indicate when the images double or when he/she is no longer able to fuse the Vectograph images. (This is the Breaking Point.)
9. Note the number/letter in the space on the blue bar at the bottom of the Vectograph. Refer to "Scoring" section of this manual.
10. Next, as you slowly slide the panels back together, ask the patient to indicate when he/she is able to see the doubled images as a single clear image again. (This is the Recovery Point.)
11. Note the number/letter in the in the space on the blue bar at the bottom of the Vectograph. Refer to "Scoring" section of this manual.
12. Repeat steps 5 – 11 until patient can achieve the doctor's recommended goal.
13. Follow any variation on these exercises as recommended by the doctor.

SCORING

- Each letter A-P on the bottom blue bar represents one diopter (Base-In) (Divergence / Relaxing).
- Each number 1-10 on the bottom blue bar represents one diopter. (Base-Out) (Convergence / Crossing).
- Each number 10-24 represents two diopters. (Base-Out) (Convergence / Crossing).
- 40 diopter range of separation available.
- Refer to the Patient Vision Therapy Record Form and record the date

- Above the diagonal line in the row labeled “Relaxing” record the Breaking Point for the highest letter on the blue bar that the patient is able to reach.
- In the same box, below the diagonal line, record the Recovery Point for the highest letter from the blue bar at which the patient is able to fuse the images as a single clear image again.
- Repeat the previous 2 steps for the row labeled “Crossing” and record the Breaking Point and Recovery Point for the highest number that the patient can reach on the blue bar at the bottom of the Vectograph.

ANSWER KEY

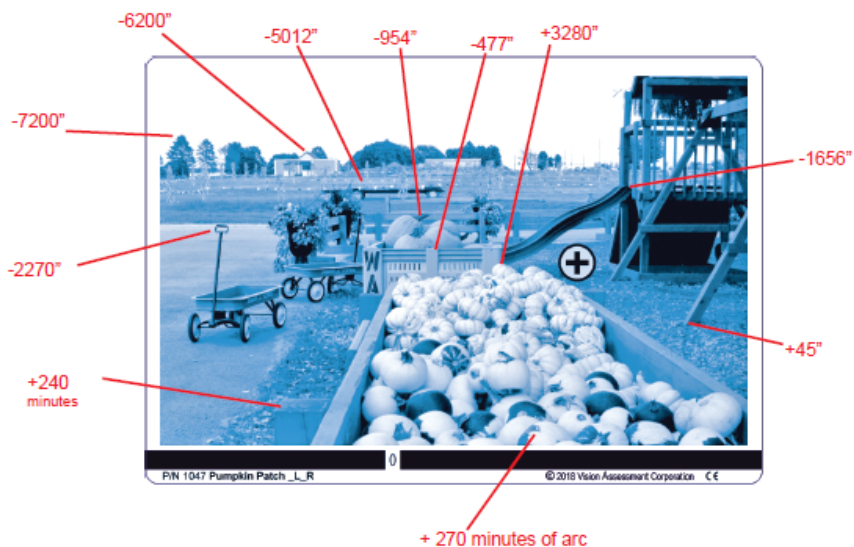

- Depth Levels measured in seconds of arc, Total Stereopsis Depth: 22,400”
- The more positive the seconds of arc > 0”, the greater the object floats **ABOVE** the Vectograph.
- The more negative the seconds of arc < 0”, the further the object floats **BELOW** the Vectograph.
- The closer the object is to 0 seconds of arc, the less the object will appear to float.


CARE/HANDLING & STORAGE

- Clean vectographic panels and guide with a soft, damp, lint-free cloth. Dampen cloth using glass cleaner or mild detergent/water.
- **CAUTION: DO NOT IMMERSE THE VECTOGRAPHIC PANELS IN WATER. DO NOT SPRAY CLEANER DIRECTLY ONTO PANELS.**
- ☀️☔ Store Vectograph in a dry place away from direct sunlight.
- Clean polarized viewers using lens cleaner and soft, lint-free cloth.
- If panels are removed from guide during cleaning, replace the panels in the guide, placing the panel with the blue bar on top of the panel with the numbers/letters. and ensuring that the plastic portion of the guide is behind the panels.

WARRANTY

- 1-year manufacturer warranty from date of purchase

Notice to User/Patient: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

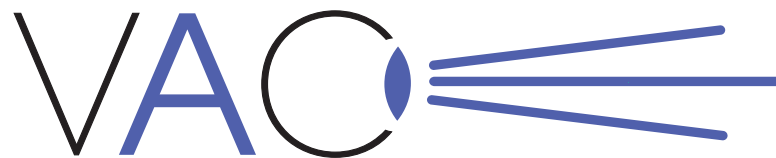

Vision
Assessment
Corporation™

Pumpkin Patch

Polarized Variable Vectograph

P/N 1047
PATIENT INSTRUCTIONS

Vision Assessment Corporation
5400 Newport Drive, Suite 3
Rolling Meadows, Illinois 60008 USA
Phone: 1 847 239 5889
Email: sales@VisionAssessment.com
Web: www.VisionAssessment.com



MDSS GmbH,
Schiffgraben 41, Hannover 30175, Germany

Manufactured in USA by Vision Assessment Corporation © 2021