

INSTRUCTIONS

Thank you for purchasing the Vision Assessment Corporation PASS Test 1, **P**ediatric **A**ssessment of **S**tereopsis with a **S**mile, P/N 1011.



PURPOSE

This depth perception test is designed for basic screening of stereopsis in children and non-readers to check how well a child's two eyes work together.

FAMILIARIZE YOURSELF WITH THE TEST

Test includes:

- 3 Cards:
 - ❖ Card **A** = DEMO
 - ❖ Card **F** = 720 seconds of arc
 - ❖ **BLANK** Card
- 1 Pair Small Pediatric Polarized Viewers
 (NOT TO BE USED AS SUNGLASSES)
- 1 Pair Medium Pediatric Polarized Viewers
 (NOT TO BE USED AS SUNGLASSES)
- 1 Instruction Manual
- 1 Measuring String (for measuring 40 cm testing distance)

Suggested Helpful Tools (NOT INCLUDED):

- Lamp (available from most ophthalmic distributors)
- Small Hand-held Mirror

TESTING CONDITIONS

1. Arrange the child's chair so that the cards are 40 cm from their eyes.
2. Choose a well-lit, glare-free area to ensure that no reflections or extraneous glare can be seen on the cards.
3. Inspect the cards to ensure that they are clean and free of smudges. Clean if necessary. Refer to "CARE/HANDLING" section at the end of this manual.

ADMINISTERING THE TEST

1. Select the appropriate size Polarized Viewers and place them on the child. (A small hand-held mirror can be used for the child to admire themselves so that they become comfortable wearing the "Magic" Viewers.)

PLEASE NOTE: IF THE CHILD WEARS PRESCRIPTION GLASSES, THE POLARIZED VIEWERS SHOULD ALWAYS BE WORN OVER THEIR NEAR CORRECTION.

2. The back of each card is labeled. Begin with Card **A** (DEMO Card). This card has a 2-dimensional Smile Face and is used for training purposes to determine task understanding.
3. Ask the child to point to or identify the Smile Face on Card **A**.
4. Next, hold Card **A** and the **BLANK** Card side by side in front of the child at a 10 degree angle (top of card tilted back towards the examiner) or place the cards on the lamp stand if available.

PLEASE NOTE: ALWAYS ENSURE THAT THE ARROW ON THE BACK OF EACH CARD IS POINTING UP. ALWAYS HOLD THE BLANK CARD AT EXACTLY THE SAME DISTANCE FROM THE CHILD AS THE OTHER CARD.

5. Hold the cards close to or touching each other without overlapping so both cards are in the child's line of sight, and ask the child to point to the Smile Face.
6. Shuffle the cards behind your back so as to randomly change the position of the Smile Face and then present it randomly either to the right or left of the **BLANK** Card.

PLEASE NOTE: AVOID CREATING A PATTERN OF PRESENTATION (i.e. R, L, R, L or R, R, L, L).

7. Repeat steps 4-6 a maximum of 5 times. The child is successful if they are able to identify the card with the Smile Face 4 of 4 or 4 of 5 presentations. The child is unsuccessful if they answer incorrectly 2 times out of a maximum of 5 presentations.
8. If the child is unsuccessful on Card **A** (DEMO), STOP testing and record the child's results "Unable."
9. If the child is successful on Card **A** they are "able" to understand the test. Proceed by replacing Card **A** with Card **F**.
10. Repeat the same procedure (steps 4-7) again now with Card **F** and the **BLANK** Card.

PLEASE NOTE: IT MAY TAKE A FEW SECONDS FOR THE CHILD TO SEE THE SMILE FACE. SUGGEST THAT THE SMILE FACE IS HIDING AND ENCOURAGE THE CHILD TO CONTINUE LOOKING FOR IT.

11. If the child is unsuccessful on Card **F**, STOP testing and record the child's results as "Card **A**" (the last card with 4 correct).
12. If the child is successful on Card **F**, record the child's results as "Card **F**."

RECORDING THE RESULTS


- Record "Refused Glasses" if the child will not wear the Polarized Viewers.
- Record "Unable" if the child is unable to point to the Smile Face in Card **A** (DEMO Card).
- Record the card for the last presentation with 4 correct responses (i.e. "Card **A**" or "Card **F**")

CARE/HANDLING & STORAGE

- Clean cards with a soft, damp, lint-free cloth. Dampen cloth using glass cleaner or mild detergent/water
- Clean Polarized Viewers using lens cleaner and soft, lint-free cloth
- **CAUTION: DO NOT IMMERSE THE CARDS IN WATER**
- **DO NOT SPRAY CLEANER DIRECTLY ONTO CARDS**
- **TO PREVENT SCRATCHING, STORE CARDS WITH FILM SIDE AGAINST FILM SIDE (FACE TO FACE)**
-  **TO AVOID FADING: STORE IN A COOL, DRY TEMPERATURE CONTROLLED ENVIRONMENT OUT OF THE HEAT, HUMIDITY & DIRECT SUNLIGHT**

WARRANTY

- 1 year manufacturer warranty from date of purchase.

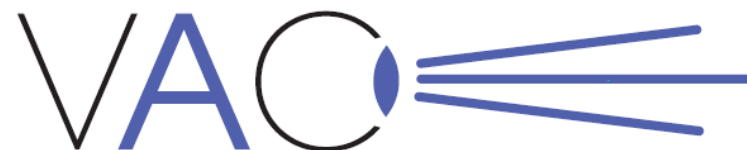

Vision
Assessment
Corporation

PASS Test 1

***Pediatric Assessment of
Stereopsis with a Smile***

P/N 1011
INSTRUCTIONS

Vision Assessment Corporation
5400 Newport Dr., Ste. 3
Rolling Meadows, IL 60008 USA
Tel.: 1 847 239 5889
sales@VisionAssessment.com
www.VisionAssessment.com

Vision Assessment Corporation would like to express its appreciation to Dr. Elise B. Ciner and the Vision in Preschoolers Study Group for their help in the design and development of this test.

Licensed from the Pennsylvania College of Optometry at Salus University.
All Rights Reserved.



MDSS GmbH,
Schiffgraben 41, Hannover 30175, Germany

 Manufactured in USA by Vision Assessment Corporation © 2009